# Veroval® Netzgerät

## Deutsch

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### Made under the control of PAUL HARTMANN AG by

Globalcare Medical Technology Co., Ltd.  
Guangdong Province, P.R.C.  
7th Building, 39 Middle Industrial Main Road, European Industrial Zone, Xiaolan Town, 528415 Zhongshan City

EC/REP Donawa Lifescience Consulting Srl  
Piazza Albania, 10  
00153 Roma  
Italia

## Veroval® AC/DC Adaptor

## English

**Operating instructions**  
**AC/DC Adaptor for HARTMANN Veroval® Blood Pressure Monitors**

### Product description

The mains part plug converts the alternative voltage of 100-240 V to a safety extra-low voltage. The device is tested according to the medical standard IEC 60601-1 and can be used with Veroval® blood pressure monitors.

### Safety and precautionary measures

- Read the operating instructions before initial use.
- Only connect the power supply unit to power sources that are suitable for this purpose. Ensure the polarity is correct.
- The power supply unit must never be exposed to rain, moisture or high heat.
- Only operate power supply unit indoors.
- Do not use the power supply unit when damaged.
- Do not open or modify the power supply unit. Repairs may only be carried out by authorised technical personnel.
- Do not cover the power supply unit. Ensure sufficient cooling. Do not expose to direct sunlight.
- Keep power supply unit and cables away from flammable items.
- Only clean with a damp cloth. Do not use any cleaning products. Do not sterilise device.

### Initial use

The power supply unit is ready for use once it is plugged into a mains socket. The power supply unit must not be used outside of the specified input values as this may lead to the device being destroyed.

Type:	LXCP12-006060BEH
Item number:	925 391
Input voltage:	100 – 240V AC, 50/60Hz
Max input current:	500mA
Output voltage:	6V DC, 600mA, only in conjunction with the Veroval® upper arm blood pressure monitor
Max output current:	600mA
 	Polarity of the DC voltage connection
Output current protection:	Protection against short circuit
No-load losses:	< 0.075 watts
Approvals:	Medical standard IEC 60601-1
Conformity:	CE compliant as per European guidelines
Operating conditions:	Ambient temperature: 0 °C to 40 °C Relative humidity: 20 – 90 <span> </span> %, non-condensing
Storage / transport conditions:	Ambient temperature: -25 °C to +70 °C Relative humidity: 20 – 90 <span> </span> %, non-condensing
Dimensions L x W x H in mm:	62 x 39 x 26 (without mains plug)

### Signs and symbols:

	Please note
	Follow instructions for use
	Labelling according to Directive 93/42/EEC on Medical Devices
	Do not use outdoors (indoor use only)
	Insulated, device class 2
	Symbol regarding electrical and electronic equipment labelling

	Donawa Lifescience Consulting Srl Piazza Albania, 10 00153 Rome Italy
---	--

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 Batch designation

 Manufacturer

 European Authorised Representative

 Dispose of packaging in an environmentally friendly manner

 Dispose of packaging in an environmentally friendly manner

 Temperature limitation

 Humidity limitation

### Warranty conditions

- Warranty claims must be asserted within the warranty period. The date of purchase must be proven by the purchase receipt.
- Within the warranty period, HARTMANN will provide a replacement free of charge for any material and manufacturing defects on the device or will repair these defects. This does not result in an extension of the warranty period.
- The device is only intended for the purpose described in these instructions for use.
- Damage that has been caused by improper use or unauthorised operations are not covered by the warranty. Accessories that are subject to wear and tear are excluded from the warranty. Claims for damages are limited to the goods value; compensation for consequential loss is expressly excluded.
- In the case of a warranty claim, please send the power supply unit with the purchase receipt to the responsible customer services department in your country either directly or via your dealer.

### Packaging contents

1 Power supply unit  
1 Operating instructions

### Information on disposal of electronic devices

This product is subject to the European Directive 2012/19/EU on Used Electrical and Electronic Equipment and is marked accordingly. Never dispose of electronic equipment in your household waste. Please obtain information about local regulations on the proper disposal of electrical and electronic products. Proper disposal protects the environment and human health.

### Contact details for customer queries

ZA HARTMANN South Africa  
Epsom Avenue, Northriding,  
2194 Johannesburg  
www.veroval.info  
Tel. +27 860 4278 6266  
phzhahelpdesk@hartmann.info

Date of revision of the text: 2017-02

### Made under the control of PAUL HARTMANN AG by

Globalcare Medical Technology Co., Ltd.  
Guangdong Province, P.R.C.  
7th Building, 39 Middle Industrial Main Road, European Industrial Zone, Xiaolan Town, 528415 Zhongshan City

EC/REP Donawa Lifescience Consulting Srl  
Piazza Albania, 10  
00153 Rome  
Italy

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**Intocmit sub supravegherea PAUL HARTMANN AG de**
Donawa Lifescence Consulting Srl Piazza Albania, 10 0